

RUNNER

Dynamic lumbar-sacral corset



Indications:

- Lumbago
- Lumbar injury
- Post-operative recovery
- Post-traumatic recovery


Available sizes	S	M	L	XL	XXL
Waist circumference from cm	60	81	91	101	111
to cm	80	90	100	110	127

Technical specifications:

- Height: back 24 cm and front 15 cm.
- Lumbar support: removable and adjustable with Velcro
- Four ribs at the back
- With its low front height, it is ideal for drivers, bikers, labourers and sportsmen

Wearing instructions (ed. 04/2010):



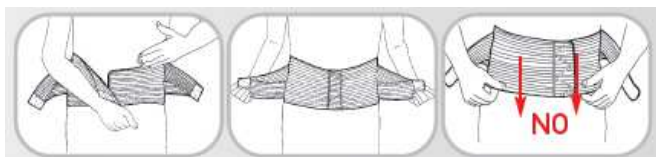


- Always wear the corset over an undervest, taking care that the inside label indicating the upper part of the corset is at the top.
- Centre the corset at the back (the four back bones must be situated on either side of the spine, two on the right and two on the left).
- Close the corset by placing the lower part on the pubic bone.
- Pull the lumbar support, positioning it on the bottom front of the corset.
- After putting on the corset, do not stretch it downwards with your hands to avoid damaging the elastic fabric. If the corset has been positioned incorrectly, open it and put it on again following the instructions.



See the wearing video:



CARE INSTRUCTIONS:

-  Hand wash in cold water with mild soap
-  Do not bleach
-  Do not iron
-  Do not dry clean
-  Do not tumble dry
- Do not wring or twist
- Dry and store away from direct heat
- Wash with velcro closed

Warnings:

Use the correct size corset and substitute it in the event of weight gain or an increase chest size.

We do not recommend wearing the corset at night.

Do not remove the inside label which contains identifying about the corset: model, size and lot number.

No significant side effects linked to correct use of the corset have been recorded.

Please note that any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority.

CE Medical Device

Compliant with MDR 2017/745

Made in Italy by highly specialized personnel.

Manufacturer:

TLM srl - Via Per Uboldo 48 - I 21040 Gerenzano (VA) - ITALY

Company with quality management system UNI EN ISO 9001 and UNI EN ISO 13485 certified by Certiquality